

How to Apply QR (Quick Relief)[®] Powder

(for minor external wounds)

Clean wound and apply only when bleeding resumes.

QR Powder needs blood to work.

For Cuts and Lacerations



1. **PEEL** back of blister.



2. **POUR** onto blood. Completely cover the wound. Occasionally upon application, a temporary warming or stinging sensation is felt.



3. **TAP & PRESS** for 30 seconds to form the protective scab and stop the bleeding.

Repeat application if needed. The protective scab is safe to leave in place until it falls off naturally. Avoid wetting the scab for one hour.

For Sliced Fingertips



1. **POUR** onto blood. Completely cover the wound. Occasionally upon application, a temporary warming or stinging sensation is felt.



2. **TAP & PRESS** for 30 seconds. Repeat application until bleeding stops.



3. **WRAP.** For added protection, wrap with gauze or bandage.

If QR Powder must be removed as determined by a healthcare professional, brush away loose powder then soak wound in sterile water until scab starts loosening from wound. Gently scrape off loosened scab with gloved finger or tongue depressor.

Warnings: FOR EXTERNAL USE ONLY. Not intended to replace stitches. Seek medical attention if wound requires stitches. Do not apply QR Powder to arterial bleeding or visibly infected wounds. Keep out of reach of children. May be harmful if inhaled or ingested. Avoid aspirating the product into the nasal or oral cavity. Avoid contact with eyes; if contact should occur, thoroughly rinse eyes with water. Seek medical attention if any irritation persists or if bleeding continues because this could be an indication of a more serious problem.

CAUTION: Store in a clean, dry place. Use by the expiration date listed. Discard if dispenser is damaged, opened, or if powder has clumped and does not flow freely. Discard contents after use, since QR Powder is guaranteed to work only if it has not been left exposed to air or humidity.

Ingredients: Hydrophilic polymer and Potassium salt.

For Cuts & Lacerations

QR QUICK RELIEF[®]

Topical Powder

**STOP BLEEDING
INSTANTLY!**

For INDUSTRIAL USE