

# How to Apply QR (Quick Relief)<sup>®</sup> Powder

(for minor external wounds) Clean wound and apply only when bleeding resumes. **QR Powder needs blood to work.**


**For Facial Lacerations, Hairline Lacerations and Other Difficult to Apply Areas.** Clean the wound and apply only when bleeding resumes. QR Powder needs blood to work.



**1. Peel** back of blister.



**2. Load Applicator.** Roll applicator to load with as much powder as possible.

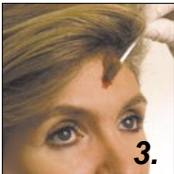


**3. Cover Wound** with as much powder as possible. Occasionally upon application, a temporary warming or stinging sensation is felt.



**4. Press.** Apply light pressure for 30 seconds to form the protective scab and stop the bleeding.

## For Nosebleeds

(Use only in the lower part of the nasal cavity). Gently blow nose to clear clots and apply only when bleeding resumes. QR needs blood to work. Do not tilt head during application. Hold breath while applying QR Powder to avoid sneezing and coughing.

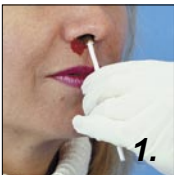

**3. Follow steps 1 and 2 above then; Roll Against Nasal Wall,** then roll applicator around the nostril. Occasionally a temporary warming or stinging sensation is felt.



**4. Pinch Nose** for 15 seconds.

Repeat application if needed. The protective scab is safe to leave in place until it falls off naturally. Avoid wetting the scab for one hour. If QR Powder must be removed as determined by a healthcare professional, brush away loose powder then soak wound in sterile water until scab starts loosening from wound. Gently scrape off loosened scab with gloved finger or tongue depressor.

**Warnings: FOR EXTERNAL USE ONLY.** Not intended to replace stitches. Seek medical attention if wound requires stitches. Do not apply QR Powder to arterial bleeding or visibly infected wounds. Keep out of reach of children. May be harmful if inhaled or ingested. Avoid aspirating the product into the nasal or oral cavity. Avoid contact with eyes; if contact should occur, thoroughly rinse eyes with water. Seek medical attention if any irritation persists or if bleeding continues because this could be an indication of a more serious problem.

**CAUTION:** Store in a clean, dry place. Use by the expiration date listed. Discard if dispenser is damaged, opened, or if powder has clumped and does not flow freely. Discard contents after use, since QR Powder is guaranteed to work only if it has not been left exposed to air or humidity.

**Ingredients:** Hydrophilic polymer and Potassium salt.

