

WOUNDSEAL™

Removal Instructions

1. If possible, do not disturb the scab. The wound is sealed and the healing process has begun. Like other scabs, it will fall off naturally.
2. If the wound site must be observed:
 - a. Rinse site with saline solution and gently remove top layer of scab with cotton-tipped swab.
 - b. Repeat saline rinse and gentle mechanical debridement as needed.
3. If the wound requires remodeling:
 - a. Rinse scab with saline solution
 - b. Remove scab with sharp debridement
 - c. Remodel wound and close.